



May 1, 2013

RE: Compatibility of Olympus and Gyrus ACMI Flexible Endoscopes and Ancillary Products with Aldahol[®] 1.8% High-level Disinfectant/Sterilant

Dear Health Care Professional:

The purpose of this letter is to inform Olympus and Gyrus ACMI users that Olympus has successfully validated the compatibility of Aldahol 1.8% High-level Disinfectant/Sterilant with Olympus and Gyrus ACMI flexible endoscopes and ancillary equipment.

The following document summarizes the compatibility of the listed Olympus and Gyrus ACMI flexible endoscopes and ancillary products with Aldahol 1.8% High-level Disinfectant/Sterilant manufactured by Alden Medical LLC and distributed by Olympus America Inc.

The conclusion of these studies is that Olympus and Gyrus ACMI endoscopes and ancillary products listed below are materially and functionally compatible with Aldahol 1.8% when used in accordance with manufacturer's instructions for room temperature (20°C) exposure and a ten minute contact time.

When performing manual high-level disinfection or sterilization using Aldahol 1.8%, avoid unnecessary long-term immersions. Consecutive reprocessing sessions using extended immersion may result in endoscope damage. Follow the instructions provided in the Olympus or Gyrus ACMI product manual for inspection of the device prior to clinical use.

Aldahol[®] 1.8% is a registered trademark of Healthpoint, Ltd.

OLYMPUS AMERICA INC.
3500 Corporate Parkway
P.O. Box 610
Center Valley, PA 18034-0610
Telephone: (484) 896-5000
www.olympusamerica.com

Page 1 of 6



OLYMPUS FLEXIBLE ENDOSCOPES

Gastrosopes / Enteroscopes	
GIF	-30 Series (All)
GIF	-40 Series (All)
GIF	-130 -1T130
GIF	-140 -1T140 -P140 -Q140 -XQ140
SIF	-Q140
GIF	-160 -1T160 -2T160 -Q160 -Q160Z -XP160 -XTQ160
GIF	-H180 -H180J -N180 -Q180 -XP180N -2TH180
GIF	-H190 -HQ190 -XP190N
SIF	-Q180
GIF	-XP240
GIF	-E Series (All)
GIF	-V
SIF	-SW

Colonoscope	
CF	-30 Series (All)
CF	-40 Series (All)
CF	-130I/L/S
PCF	-130I/L
CF	-140I/L/S -1T140I/L -2T140I/L -Q140I/L
PCF	-140I/L
CF	-Q160DI/L -2T160L -Q160I/L/S -Q160ZI/L
PCF	-160AI/AL -S
CF	-H180AI/AL -Q180AI/AL -H180DI/DL
CF	-H190L/I -HQ190L/I
PCF	-H180AI/AL -Q180AI/AL
PCF	-H190L/I -PH190L/I
CF	-Q240ZI/ZL -2T200 -2T160L/I
CF	-E Series (All)
OSF	-3 -4 -V60

Bronchoscopes	
BF	-10 Series (All)
BF	-20 -1T20 -1T20D -20D -3C20 -N20 -P20 -P20D -XT20
BF	-30 -1T30 -3C30 -P30 -XT30
BF	-40 -1T40 -3C40 -P40 -XP40 -XT40
BF	-1T60 -MP60 -P60 -XP60
BF	-160 -1T160 -3C160 -MP160F -P160 -XP160F -XT160
BF	-1T180 -1TQ180 -P180 -Q180 -Q180-AC
BF	-H190 -Q190 -1TH190
BF	-200 -1T200 -P200
BF	-240 1T240 -6C240 P240

Duodenoscopes	
JF/TJF	-30 Series (All)
JF/TJF	-40 Series (All)
JF	-130
TJF	-130
JF	-140/F/R
TJF	-140F
TJF	-160F/VF
PJF	-160
TJF	-Q180V
JF/TJF	-E Series (All)
JF	-VJF/TJF -E Series (All)
PJF	-7.5JF -V -V2
JF	-V2PJF -7.5

OLYMPUS AMERICA INC.
 3500 Corporate Parkway
 P.O. Box 610
 Center Valley, PA 18034-0610
 Telephone: (484) 896-5000
 www.olympusamerica.com



ACCESSORIES FOR GI ENDOSCOPES

Accessory Name	Model	Accessory Name	Model	Accessory Name	Model
Water container	MAJ-901	Gas tube	MA-238	Distal Cover	MAJ-311
	MAJ-902		MD-411		MAJ-411
	MD-412	Air/Water valve	MH-438	Water Resistant Cap	MAJ-583
	MD-431		MB-196		MAJ-942
	MH-884		MD-227		MAJ-639
	Gas/water valve	MAJ-521	Suction Valve	MH-443	Channel Plug
MD-410		MB-197		AW Channel Cleaning Adapter	
MB-878			MD-192	Channel Cleaning Brush	BW-20T
Auxiliary water tube	MAJ-855	Suction Cleaning Adaptor	MH-856	Injection tube	MH-946
Channel connection tube	MAJ-420	Washing tube	MH-974	Biopsy Valve	MAJ-419
		Cleaning Adapter for Air/Water Channel	MB-107	Channel Opening Cleaning Brush	MH-507
MAJ-579	Position Detecting Probe		MAJ-1300		
Right/Left Angulation knob attachment	MAJ-1072				

OLYMPUS CAMERA HEADS, VIDEO ADAPTERS/COUPLERS

Product Name	Model
OTV-S5 Series	MH-972D/I/N MH-973D/I/N
OTV-S6H	-N -D -1N -1D -1F -1L -NA -1NA
OTV-S6H-FA	-E -Q -S
OTV-S6H-1FA	-E
ImagTrac	-H
OTV-S7H	-N -1N -1D -NA -1NA -VA
OTV-S7H-1D	-F08E -L08E
OTV-S7H-NA	-10E -10Q -12E -12Q
OTV-S7H-1NA	-10E -10Q -12E -12Q
OTV-S7H-FA	-E/Q

Product Name	Model
Video Coupler	MH-999E/S/Q
Video Adapter	AR -T10 -10E -10S -10Q -T12 -12E -12S -12Q -TF08E -L08E -TL12S -TZ2 -SX12 -E -S -Q -ZX20 -FX08E -LX08
	A10 -T1 T2
ImagTrac	MAJ-595 -596
OTV-SC	MAJ-554
OTV-S7ProH-HD Series	-10E -10Q -12E -12Q -L08E
OTV-SX2 Series	MAJ-387N/I

OLYMPUS AMERICA INC.
 3500 Corporate Parkway
 P.O. Box 610
 Center Valley, PA 18034-0610
 Telephone: (484) 896-5000
www.olympusamerica.com



ULTRASONIC ENDOSCOPES

Product Name	Model
Mechanical Ultrasonic Endoscopes	GF -UM20 -UM30P -UM130 -UM160 -UMQ130
	CF -UM20
	JF -UM20
	TGF -UC180J
Electric Ultrasonic Endoscopes	GF -UC30P -UC140P-DO5 -UC140P-AL5 -UC160P-OL5 -UC160P-AT8 -UCT140-AL5 -UCT140-DO5 -UCT160-OL5 -UCT160-AT8 -UE160-AL5 -UCT180
	BF -UC160F-OL8 -UC180F
Ultrasonic Probe	UM -2R -3R -BS20-26R -S20-17S† -G20-29R -S20-20R -S30-20R -S30-25R -DG20-31R -DP12-25R -DP20-25R
	RU -12M-R1 -75M-R1
	MH -908

†The distal end of the UM-S20-17S ultrasonic probe expands following long-term continuous soaking. The stretched sheath reverts to the initial length after drying. Do not soak the probe in the disinfectant solution for extended times exceeding the labeling condition. Inspection of the probe must be performed prior to clinical use according to the procedure in the Olympus product manual. In the case that soaking duration exceeds the labeling condition, do not use the probe immediately after reprocessing. Allow the sheath to revert to original size. This phenomenon does not affect functionality of the device.

ACCESSORIES FOR ULTRASONIC PRODUCTS

Product Name	Model
Water resistant Cap	MAJ-896
	MAJ-1057
	MAJ-1174
	MH-244
Balloon sheath connector for Ultrasonic Endoscopes	MD-688
	MD-756
Probe holder	MH-245
Biopsy valve	MAJ-853

OLYMPUS AMERICA INC.
 3500 Corporate Parkway
 P.O. Box 610
 Center Valley, PA 18034-0610
 Telephone: (484) 896-5000
www.olympusamerica.com

OLYMPUS AND GYRUS ACMI SURGICAL FLEXIBLE ENDOSCOPES

Cystoscopes
CYF -200
CYF -240 /A
CYF -V -V2 -V2R -VA -VA2 -VH -VHR -VHA
CYF -2 -3 -4 -4A -5 -5A 5R
ACN -2 -2BA -2T -2TBA
ICN -0564 -0565 2 2BA
Hysteroscopes
HYF -V
HYF -1T -P -XP
Ureteroscopes
URF -P2 -P3 -P5 -P6 -P6R
URF -V
AUR -7 -735
DUR -8 -8BA -8E -8EBA
DUR -8ULTRA -8ULTRA-BA
DUR -D -DBA
Choledochoscopes
CHF -CB20 -P20 -P20Q
CHF -CB30L/S
CHF -P60
CHF -V
CHF -B20 -BP30

Laparo-Thoraco Scopes
LTF -160
LTF -240
LTF -260
LTF -V -V2 -V3 -VH -VP -VP-S
Tracheal Intubation Scopes
LF -V
LF -1 -2 -P -T -DP -TP -GP
Airway Mobilescopes
MAF -GM
MAF -TM
Needles Fiberscopes
MSS-28CS -301 -307
Rhino-Laryngo Scopes
ENF -200
ENF -V -V2 -VQ -VT -VT2 -V3 -VH
ENF -GP -L3 -P2 -P3 -P4 -T3 -XP
PEF -V
Ultrathin Fiberscopes
PF -14 -22 -28



ACCESSORIES FOR SURGICAL PRODUCTS

Product Name	Model
Miniature light source	MAJ-524
	MAJ-922
Suction valve	MAJ-207
ETO cap	MB-156
Forces/Irrigation plug	MAJ-891
Suction connector cleaning brush	BW-15SH
Channel cleaning brush	BW-7B
Forces/Irrigation plug	BW-15B
Mouthpiece	MA-651
Water-resistant cap	MAJ-1214
Sterilization cap	MAJ-1538

NON-OLYMPUS AND NON-GYRUS ACMI REPAIRS

Neither Olympus nor Gyrus ACMI can guarantee that products repaired by a non-Olympus or non-Gyrus ACMI facility will be compatible with Aldahol 1.8%. As a result, this compatibility statement is invalid for products repaired by a non-Olympus or non-Gyrus ACMI facility. Moreover, if a device is NOT repaired by the original product manufacturer, the corresponding product warranty may be voided. In the event that your products have been repaired by such a facility, please contact the repair facility directly for compatibility information.

WARRANTY

Nothing contained in this letter alters, extends, or modifies in any way the authorized Olympus warranty or Gyrus ACMI's Limited Express Warranty applicable to each device or instrument.

If you have any questions regarding the compatibility of Olympus or Gyrus ACMI products with Aldahol 1.8%, contact your local Olympus or Gyrus ACMI sales representative or the Olympus Technical Assistance Center at 1-800-848-9024 (United States) or 1-800-387-0437 ext 703211 (Canada) or Gyrus ACMI Technical Support at (888) 524-7266.

Thank you.

Sincerely,

Mary Ann Drosnock
Manager, Infection Control
Clinical Affairs Department
Olympus America, Inc.
Maryann.drosnock@olympus.com

OLYMPUS AMERICA INC.
3500 Corporate Parkway
P.O. Box 610
Center Valley, PA 18034-0610
Telephone: (484) 896-5000
www.olympusamerica.com