

## Cleaning, Disinfection and Sterilization list for Ultrasound System



This list shows the disinfectants that can be used in the system of Ultrasound system. The explanation of C, N and U in the list is as follows.

Item	Definition
C:	Compatible
C*:	Compatible. Soak the cutting cotton or gauze with alcohol and squeeze hard, and then wipe it lightly.
C*2:	Compatible. Do not wipe it strongly.
C*3:	Compatible. Soak with a small amount of liquid in the cloth and then wipe it lightly.
N:	No (do not use)
U:	Unknown/Untested

### CAUTION

The disinfectants listed in this list guarantee resistance to the system. However, "cleaning / disinfecting / sterilizing effects" is not guaranteed. For "cleaning / disinfecting / sterilizing effects", please refer to each disinfectant's manufacturer's website, instruction manual, etc.

- Each parts of system

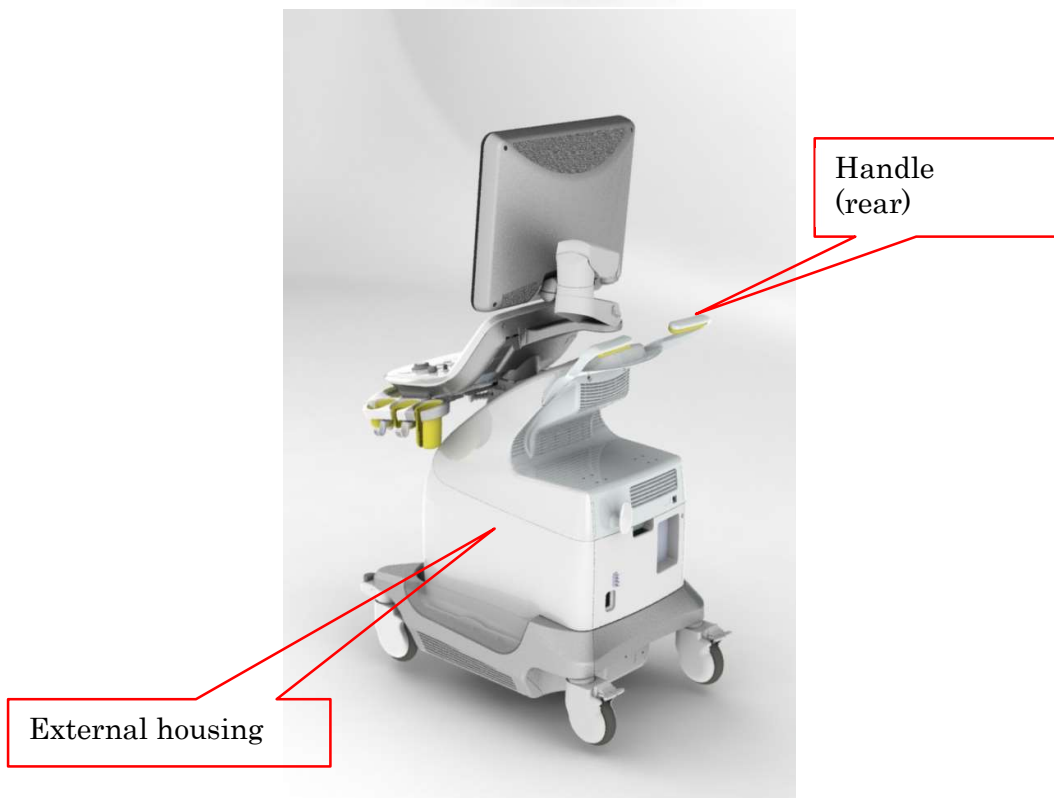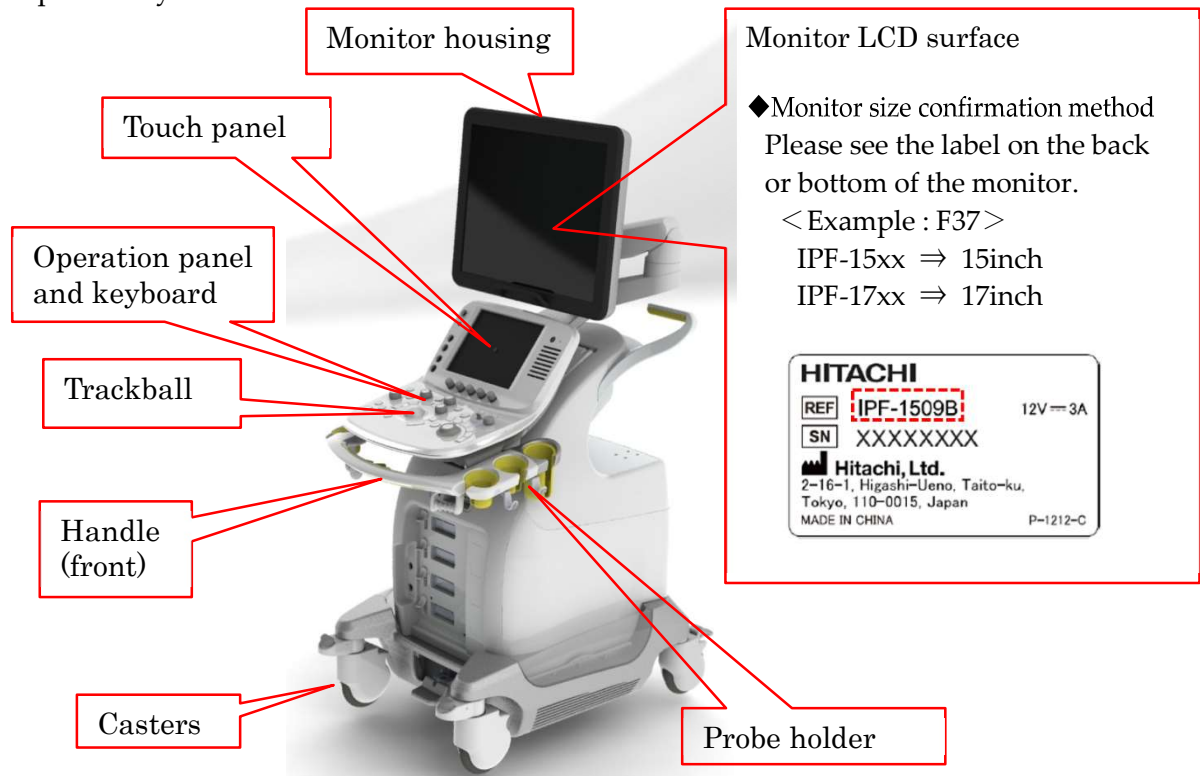

The product names, etc. described in this material are registered trademarks or trademarks of their respective companies.

Disinfectant	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	Prosound F75								
						19 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	
Sani-Cloth® HB	USA	PDI	Wipe	disinfectant	Quaternary ammonium (alcohol free)	U	C	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfectant	Quaternary ammonium (low alcohol)	U	C	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfectant	Isopropanol	U	C	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfectant	Quaternary ammonium	U	C	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfectant	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe	disinfectant	Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid	disinfectant	Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfectant	Propanol	U	C	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfectant	Chlorine dioxide	C	C	C	C	C	C	C	C	C
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	disinfectant	Didecyldimethylammonium chloride	C	C	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfectant	Low alcohol	C	C	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe	disinfectant	Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfectant	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfectant	Ethanol	C	C	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfectant	Propanol	C	C	C	C	C	C	C	C	C
Methyl alcohol max. 80% vol.	-		Liquid	disinfectant		C	C	C	C	C	C	C	C	C



Disinfectant	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	SSD-5000											SSD-5500												
						15 Inch monitor CRT surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Touch panel	Probe holder	Handle	Casters	External housing	15 Inch monitor CRT surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Touch panel	Probe holder	Handle	Casters	External housing		
Sani-Cloth® HB	USA	PDI	Wipe	disinfectant	Quaternary ammonium (alcohol free)	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Sani-Cloth® Plus	USA	PDI	Wipe	disinfectant	Quaternary ammonium (low alcohol)	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Super Sani-Cloth®	USA	PDI	Wipe	disinfectant	Isopropanol	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfectant	Quaternary ammonium	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfectant	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Accel PREvention Wipes	Canada	AHP	Wipe	disinfectant	Accelerated hydrogen peroxide	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Accel PREvention RTU	Canada	AHP	Liquid	disinfectant	Accelerated hydrogen peroxide	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfectant	Propanol	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Tristel Duo	EU	Tristel Company	foam	disinfectant	Chlorine dioxide	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	disinfectant	Didecyldimethylammonium chloride	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfectant	Low alcohol	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
セイフキープ	JPN	KAO (花王)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe	disinfectant	Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfectant	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	U	U	U	C	U	C	C	U	U	C	C	C	U	U	U	C	U	C	C	U
Ethyl alcohol max. 80% vol.	-		Liquid	disinfectant	Ethanol	U	C	C	C	U	U	U	C	C*	C	C	U	U	C	C	C	U	U	U	C	C*	C	C	U
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfectant	Propanol	U	C	C	C	U	U	U	C	C*	C	C	U	U	C	C	C	U	U	U	C	C*	C	C	U
Methyl alcohol max. 80% vol.	-		Liquid	disinfectant		U	C	C	C	U	U	U	C	C*	C	C	U	U	C	C	C	U	U	U	C	C*	C	C	U

Disinfectant	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	SSD-Alpha5											SSD-Alpha10										
						Monitor LCD surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Probe holder cover	Probe holder	Handle	Casters	External housing	Monitor surface	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing			
Sani-Cloth® HB	USA	PDI	Wipe	disinfectant	Quaternary ammonium (alcohol free)	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Sani-Cloth® Plus	USA	PDI	Wipe	disinfectant	Quaternary ammonium (low alcohol)	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Super Sani-Cloth®	USA	PDI	Wipe	disinfectant	Isopropanol	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfectant	Quaternary ammonium	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfectant	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Accel PREvention Wipes	Canada	AHP	Wipe	disinfectant	Accelerated hydrogen peroxide	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Accel PREvention RTU	Canada	AHP	Liquid	disinfectant	Accelerated hydrogen peroxide	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfectant	Propanol	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfectant	Ammonium	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Tristel Duo	EU	Tristel Company	foam	disinfectant	Chlorine dioxide	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	disinfectant	Didecylidimethylammonium chloride	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfectant	Low alcohol	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
セイフキープ	JPN	KAO (花王)	Wipe	disinfectant	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe	disinfectant	Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	
Sodium hypochlorite	-		Liquid	disinfectant	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	U	U	C	U	U	C	U	C	C	U	C	C	U	C	U	C	C	
Ethyl alcohol max. 80% vol.	-		Liquid	disinfectant	Ethanol	U	C	C	C	U	U	C	C	C*	C	C	U	C	C	U	C	C	U	C	C*	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfectant	Propanol	U	C	C	C	U	U	C	C	C*	C	C	U	C	C	U	C	C	U	C	C*	C	C
Methyl alcohol max. 80% vol.	-		Liquid	disinfectant		U	C	C	C	U	U	C	C	C*	C	C	U	C	C	U	C	C	U	C	C*	C	C