



Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the ISM nStream and VaultStream platform. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the nStream G3, nCareRX/MX, EasyLink Router, Thin Client, Status Board Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Product: nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the Status Board Engine.

Date: December 12, 2017

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

December 2017 Updates		
KB Number	Platform	Title
KB4054518	nStream G3	Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB890830	nStream G3	Windows Malicious Software Removal Tool - December 2017
KB4054518	VaultStream	2017-12 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4052978	VaultStream	Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based Systems
KB890830	VaultStream	Windows Malicious Software Removal Tool x64 - December 2017
KB915597	VaultStream	Definition Update for Windows Defender - (Definition 1.259.172.0)
KB4051956	ELR v8.0	Update for WES09 and POSReady 2009
KB4021915	ELR v8.0	June, 2017 Security Only Update for .NET Framework 4 on WES09 and POSReady 2009
KB4049068	ELR v8.0	Update for WES09 and POSReady 2009
KB4054521	nCareRX/MX/GX-SBE	2017-12 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4054518	nCareRX/MX/GX-SBE	2017-12 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4051956	nCareRX/MX/GX-SBE	Update for Windows Embedded Standard 7 for x64-based Systems
KB4049068	nCareRX/MX/GX-SBE	Update for Windows Embedded Standard 7 for x64-based Systems
KB4049016	nCareRX/MX/GX-SBE	2017-11 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7 on Windows 7 and Server 2008 R2 for x64 - Contains -KB4019990 and KB4040980
KB4055038	nCareRX/MX/GX-SBE	2017-11 Update for Windows Embedded Standard 7 for x64-based Systems
KB4054521	ThinClient-ELR v8.1	2017-12 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4055038	ThinClient-ELR v8.1	2017-11 Update for Windows Embedded Standard 7 for x86-based Systems

KB4054518	ELR v8.1	2017-12 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
PHP Updates		
None		
Not Approved Updates:		
December 2017 Updates		
KB Number	Platform	Title
KB4021915	ELR v8.0	June, 2017 Security Only Update for .NET Framework 4 on WES09 and POSReady 2009
KB4054518	ThinClient	2017-12 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems

For further questions concerning this notification please call Technical Support at (866) 376-6800.