



Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the ISM nStream and VaultStream platform. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the nStream G3, nCareRX/MX, EasyLink Router, Thin Client, Status Board Engine, MedPresence Stand-Alone, All-In-One Client, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Product: nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the Status Board Engine.

Date: July 13, 2018

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

July 2018 Updates		
KB Number	Platform	Title
nStream v5.4 Updates		
KB2952664	nStream	Update for Windows 7
KB4284826	nStream	2018-06 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB4338818	nStream	2018-07 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB4340556	nStream	2018-07 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7
KB890830	nStream	Windows Malicious Software Removal Tool - July 2018
VaultStream v5.3, 8.0.x, v9.0.x Updates		
KB4099633	VaultStream	2018-05 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7 and Server 2008 R2 for x64
KB4103768	VaultStream	Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based Systems
KB4284826	VaultStream	2018-06 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB890830	VaultStream	Windows Malicious Software Removal Tool x64 - June 2018
KB890830	VaultStream	Windows Malicious Software Removal Tool x64 - July 2018
KB4339093	VaultStream	Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based Systems
KB4338818	VaultStream	2018-07 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4340556	VaultStream	2018-07 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
EasyLink Router v8.0 Updates		
KB4134651	ELR v8.0	2018-05 Security Update for WES09 and POSReady 2009 for x86-based Systems

KB4230467	ELR v8.0	2018-06 Security Update for Windows Server 2008 for x86-based Systems
KB4293928	ELR v8.0	2018-06 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4294413	ELR v8.0	2018-06 Security Update for Windows Server 2008 for x86-based Systems
KB4291391	ELR v8.0	2018-07 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4134651	ELR v8.0	2018-07 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4339854	ELR v8.0	2018-07 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4339291	ELR v8.0	2018-07 Security Update for WES09 and POSReady 2009 for x86-based Systems
nCareRX v8.0 and v8.1, nCareMX/nStreamGX v8.0, v9.0, and Status Board Engine, 8.0 and v8.1 Updates		
KB4284826	nCare-SBE	2018-06 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems pciclearstalecache_fdc5fc21af7572c604f50e0e7f9f7a6c465835b4.exe
KB4284867	nCare-SBE	2018-06 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4291493	nCare-SBE	2018-06 Preview of Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1 on Windows 7 and Server 2008 R2 for x64 windows6.1-kb4095874-x64_13904512d1614ba1a336061e90f7078becf3be5a.msx
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msx
KB4338823	nCare-SBE	2018-07 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4340004	nCare-SBE	2018-07 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64 windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msx
		windows6.1-kb4338612-x64_84fb0e66e8decff609beeed94fa73887b88155c6.msx
KB4340556	nCare-SBE	2018-07 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64 windows6.1-kb4338423-x64_58b34908c283746456dd4b2b8c8b4d8e6b98fcf8.msx
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msx
KB4338818	nCare-SBE	2018-07 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems pciclearstalecache_7b1e71fc5a81cd30e0a46338caa6fb1d2880f2c2.exe
HP Thin Client v8.0 and EasyLink Router v8.1 Updates		
KB4284867	ThinClient-ELR v8.1	2018-06 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4338823	ThinClient-ELR v8.1	2018-07 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4338818	ThinClient-ELR v8.1	2018-07 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems pciclearstalecache_7e5c56007b7cac1057d85df3261a8a31b8d48ea9.exe
MedPresence Stand-Alone v9.0.x and All-In-One Client (works with v9.0.x and 10.x)		
KB4132216	MedPresence-All-In-One	2018-05 Update for Windows 10 Version 1607 for x64-based Systems
KB4284880	MedPresence-All-In-One	2018-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems

KB4284880	MedPresence-All-In-One	2018-06 Delta Update for Windows 10 Version 1607 for x64-based Systems
KB4287903	MedPresence-All-In-One	2018-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4284833	MedPresence-All-In-One	2018-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4338814	MedPresence-All-In-One	2018-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4338814	MedPresence-All-In-One	2018-07 Delta Update for Windows 10 Version 1607 for x64-based Systems
KB4338832	MedPresence-All-In-One	2018-07 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
ES4K v10.0.x - BlizCon		
KB4284880	BlizCon	2018-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4287903	BlizCon	2018-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4345418	BlizCon	2018-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4338832	BlizCon	2018-07 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4338814	BlizCon	2018-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
ES4K v10.0.x - nCareRX2		
KB4284880	nCareRX2	2018-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4287903	nCareRX2	2018-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4338832	nCareRX2	2018-07 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based System
ES4K v10.0.x - LSSB2		
KB4284880	LSSB2	2018-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4287903	LSSB2	2018-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4345418	LSSB2	2018-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4338832	LSSB2	2018-07 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4338814	LSSB2	2018-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
None		

For further questions concerning this notification please call Technical Support at (866) 376-6800.