

## Microsoft® Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®/nStream®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite an EasySuite 4K, nCare GX/MX/RX, nStream G3, EasyLink Router, Thin Client, LiveStream StatusBoard Engine, MedPresence Stand-Alone, All-In-One Client, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

**Products:** EasySuites, nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

**Date:** July 10, 2019

### **Critical Security Bulletins (Obtained from Microsoft):**

**NOTE:** Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

### **Approved Updates:**

July 2019 Updates		
KB Number	Platform	Title
<b>nStream v5.4 Updates</b>		
KB4503292	nStream	2019-06 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
<b>VaultStream v5.3, 8.0.x, v9.0.x Updates – Windows® Server 2008</b>		
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - July 2019
KB4507449	VaultStream 2008	2019-07 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4507420	VaultStream 2008	2019-07 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
KB915597	VaultStream 2008	Definition Update for Windows Defender Antivirus - (Definition 1.297.741.0)
<b>VaultStream v10.x Updates – Windows® Server 2012</b>		
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - July 2019
KB4507422	VaultStream 2012	2019-07 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB4507448	VaultStream 2012	2019-07 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4504418	VaultStream 2012	2019-07 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
<b>VaultStream v10.x Updates – Windows® Server 2016</b>		
KB4509091	VaultStream 2016	2019-07 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4507460	VaultStream 2016	2019-07 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - July 2019
<b>EasyLink Router v8.0 Updates</b>		
None		
<b>nCare RX v8.0 and v8.1, nCare MX/ nStream GX v8.0, v9.0, and Status Board Engine, 8.0 and v8.1 Updates</b>		

KB4503865	nCare-SBE	2019-06 Preview of Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
		windows6.1-kb4495606-x64_689f245559ffc28ba33ad3675287eff1a014174b.msu
KB4508772	nCare-SBE	2019-06 Update for Windows Embedded Standard 7 for x64-based Systems
KB4507456	nCare-SBE	2019-07 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4507449	nCare-SBE	2019-07 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4507411	nCare-SBE	2019-07 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4506976-x64_1e2c90690ff689ae81cf14155edac3bca8790ab0.msu
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
KB4507420	nCare-SBE	2019-07 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.ms
		windows6.1-kb4507004-x64_a7d988d65422c8cfbbbed141c95aeea37a8743167.msu
KB4507004	nCare-SBE	2019-07 Security and Quality Rollup for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64
KB4506976	nCare-SBE	2019-07 Security Only Update for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64
<b>HP Thin Client v8.0 and EasyLink Router v8.1 Updates</b>		
KB4508772	Thin Client v8.0-ELR v8.1	2019-06 Update for Windows Embedded Standard 7 for x86-based Systems
KB4507456	Thin Client v8.0-ELR v8.1	2019-07 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4507449	Thin Client v8.0-ELR v8.1	2019-07 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
<b>nCare MX v10.1.2</b>		
KB4507460	nCare MX	2019-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4509091	nCare MX	2019-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>MedPresence Stand-Alone v9.0.x and All-In-One Client (works with v9.0.x and 10.x)</b>		
None		
<b>EasySuite 4K v10.1.2   EasySuite 4K Controller (a.k.a. BlizCon)</b>		
KB4507460	EasySuite 4K Controller	2019-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4503308	EasySuite 4K Controller	2019-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4509091	EasySuite 4K Controller	2019-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems



**Image Stream Medical**

One Monarch Drive Littleton, MA 01460 U.S.A.  
[www.olympusamerica.com/olympus-systems-integration](http://www.olympusamerica.com/olympus-systems-integration)

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas

<b>EasySuite 4K v10.1.2   EasySuite 4K Video Manager (a.k.a. nCare RX2)</b>		
KB4507460	EasySuite 4K Video Manager	2019-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4503308	EasySuite 4K Video Manager	2019-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4509091	EasySuite 4K Video Manager	2019-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>EasySuite 4K v10.0.x – LiveStream StatusBoard 2</b>		
KB4509091	LSSB2	2019-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4507460	LSSB2	2019-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>PHP Updates</b>		
5.6.40 and 7.1.30		

For further questions concerning this notice, please call Technical Support at (866) 376-6800



**Image Stream Medical**

One Monarch Drive Littleton, MA 01460 U.S.A.  
[www.olympusamerica.com/olympus-systems-integration](http://www.olympusamerica.com/olympus-systems-integration)

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas