

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®/nStream®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite an EasySuite 4K, nCare GX/MX/RX, nStream G3, EasyLink Router, Thin Client, LiveStream StatusBoard Engine, MedPresence Stand-Alone, All-In-One Client, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites, nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: June 12, 2019

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

June 2019 Updates		
KB Number	Platform	Title
nStream v5.4 Updates		
KB4503292	nStream	2019-06 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB890830	nStream	Windows Malicious Software Removal Tool - June 2019
VaultStream v5.3, 8.0.x, v9.0.x Updates – Windows® Server 2008		
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - June 2019
KB4503292	VaultStream 2008	2019-06 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB915597	VaultStream 2008	Definition Update for Windows Defender Antivirus (Definition 1.295.409.0)
VaultStream v10.x Updates – Windows® Server 2012		
KB4503276	VaultStream 2012	2019-06 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4503308	VaultStream 2012	2019-06 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - June 2019
KB3013816	VaultStream 2012	Update for Windows Server 2012 R2
KB3013769	VaultStream 2012	Update for Windows Server 2012 R2
VaultStream v10.x Updates – Windows® Server 2016		
KB4505052	VaultStream 2016	2019-05 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - June 2019
KB4503267	VaultStream 2016	2019-06 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4503537	VaultStream 2016	2019-06 Servicing Stack Update for Windows Server 2016 for x64-based Systems
EasyLink Router v8.0 Updates		
KB4501226	ELR v8.0	2019-06 Update for POSReady 2009 for x86-based Systems
nCare RX v8.0 and v8.1, nCare MX/ nStream GX v8.0, v9.0, and Status Board Engine, 8.0 and v8.1 Updates		

KB4489486	nCare MX-SBE	Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4488662-x64_6c483c4681671fe41682a4a6a8750cc6cb170a54.msu
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
KB4488669	nCare MX-SBE	Update for .NET Framework 4.5.2 on Windows 7 and Server 2008 R2 for x64
KB4499175	nCare MX-SBE	2019-05 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
		windows6.1-kb4499175-x64_3704acfff45ddf163d8049683d5a3b75e49b58cb.msu
		pciclearstalecache_d243a607b50db10ed50f03cff570498018c61a59.exe
KB4503292	nCare RX/MX-SBE	2019-06 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4503269	nCare RX/MX-SBE	2019-06 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
HP Thin Client v8.0 and EasyLink Router v8.1 Updates		
KB4499175	Thin Client v8.0-ELR v8.1	2019-05 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
		pciclearstalecache_7f0c5cff499688bc34fdf52d5a6af3d97881d801.exe
		windows6.1-kb4499175-x86_6f1319c32d5bc4caf2058ae8ff40789ab10bf41b.msu
KB4503292	Thin Client v8.0-ELR v8.1	2019-06 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
KB4503269	Thin Client v8.0-ELR v8.1	2019-06 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
nCare MX v10.1.2		
KB4505052	nCare MX	2019-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4499177	nCare MX	2019-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4503537	nCare MX	2019-06 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4503267	nCare MX	2019-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4503308	nCare MX	2019-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4346087	nCare MX	2019-02 Update for Windows 10 Version 1607 for x64-based Systems
MedPresence Stand-Alone v9.0.x and All-In-One Client (works with v9.0.x and 10.x)		
KB4503537	MedPresence SA, and All-In-One Client	2019-06 Service Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4503267	MedPresence SA, and All-In-One Client	2019-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4503308	MedPresence SA, and All-In-One Client	2019-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB890830	MedPresence SA, and All-In-One Client	Windows Malicious Software Removal Tool x64 - June 2019
EasySuite 4K v10.1.2 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4503537	EasySuite 4K Controller	2019-06 Service Stack Update for Windows 10 Version 1607 for x64-based Systems



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas

KB4503267	EasySuite 4K Controller	2019-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4503308	EasySuite 4K Controller	2019-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4493470	EasySuite 4K Controller	2019-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4346087	EasySuite 4K Controller	2019-02 Update for Windows 10 Version 1607 for x64-based Systems
KB4493478	EasySuite 4K Controller	2019-04 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.1.2 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4503537	EasySuite 4K Video Manager	2019-06 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4503308	EasySuite 4K Video Manager	2019-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4503267	EasySuite 4K Video Manager	2019-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4346087	EasySuite 4K Video Manager	2019-02 Update for Windows 10 Version 1607 for x64-based Systems
KB4493470	EasySuite 4K Video Manager	2019-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4493478	EasySuite 4K Video Manager	2019-04 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.0.x – LiveStream StatusBoard 2		
KB4503537	LSSB2	2019-06 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4503308	LSSB2	2019-06 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4503267	LSSB2	2019-06 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.30		

For further questions concerning this notice, please call Technical Support at (866) 376-6800



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas