

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates approved for the v8.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: December 9, 2020

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813, or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

December 2020 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows® Server 2012		
KB4592484	VaultStream 2012	2020-12 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
VaultStream v10.x Updates – Windows® Server 2016		
KB4593226	VaultStream 2016	2020-12 Cumulative Update for Windows Server 2016 for x64-based Systems
nCare MX v10.2.2		
KB4593226	nCare MX	2020-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4593226	EasySuite 4K Controller	2020-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4593226	EasySuite 4K Video Manager	2020-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.3 – LiveStream StatusBoard 2		
KB4593226	LiveStream StatusBoard 2	2020-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
MedPresence MedPresence Tower v10.3		
KB4593226	MedPresence Tower	2020-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.33 (7.3.25 available upon request for v10.2.2 VaultStreams only)		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024