

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., EasySuite™, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite HD and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: August 10, 2021

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

August 2021 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows® Server 2012		
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.92
KB5005076	VaultStream 2012	2021-08 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB5004873	VaultStream 2012	2021-08 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
KB2565063	VaultStream 2012	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package
VaultStream v10.x Updates – Windows® Server 2016		
KB5005043	VaultStream 2016	2021-08 Cumulative Update for Windows Server 2016 for x64-based Systems
KB5004752	VaultStream 2016	2021-08 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.92
VaultStream v10.4 Updates – Windows® Server 2019		
KB5005030	VaultStream 2019	2021-08 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems
KB5004414	VaultStream 2019	2021-07 Cumulative Update Preview for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64
KB5004308	VaultStream 2019	2021-07 Cumulative Update Preview for Windows Server 2019 (1809) for x64-based Systems
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.345.338.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.345.275.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.343.1365.0)
KB890830	VaultStream 2019	Windows Malicious Software Removal Tool x64 - v5.92
KB5005112	VaultStream 2019	Security Update for Microsoft Windows

KB5004332	VaultStream 2019	Update for Microsoft Windows
KB5004424	VaultStream 2019	Update for Microsoft Windows
KB5005030	VaultStream 2019	2021-08 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems
nCare MX v10.x		
KB5005043	nCare MX	2021-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5004752	nCare MX	2021-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
ES4K v10.x ES4K Controller (a.k.a. BlizCon)		
KB5005043	ES4K Controller	2021-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5004752	ES4K Controller	2021-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
ES4K v10.x ES4K Video Manager (a.k.a. nCare RX2)		
KB5005043	ES4K Video Manager	2021-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5004752	ES4K Video Manager	2021-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
ES4K v10.x – LiveStream StatusBoard 2		
KB5004870	LiveStream StatusBoard 2	2021-08 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64
KB5005030	LiveStream StatusBoard 2	2021-08 Cumulative update for Windows 10 Version 1809 for x64-based Systems
KB5005112	LiveStream StatusBoard 2	2021-08 Servicing Stack Update for Windows 10 Version 1809 for x64-based Systems
MedPresence Tower v10.3		
KB5005043	MedPresence Tower	2021-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5004752	MedPresence Tower	2021-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
PHP Updates		
7.1.33 and 7.4.16 – shipping versions. 7.3.29 is available upon request for v10.2.2 VaultStreams only. 7.4.21 is available upon request for v10.4 VaultStreams only.		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024