



Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite HD and ES4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

Products: EasySuite HD, ES4K and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: October 12, 2021

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

| October 2021 Updates | | |
|---|------------------|---|
| KB Number | Platform | Title |
| VaultStream v10.x Updates – Windows® Server 2012 | | |
| KB5006714 | VaultStream 2012 | 2021-10 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems |
| KB890830 | VaultStream 2012 | Windows Malicious Software Removal Tool x64 - v5.94 |
| KB5006763 | VaultStream 2012 | 2021-10 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64 |
| VaultStream v10.x Updates – Windows® Server 2016 | | |
| KB5006065 | VaultStream 2016 | Update for Microsoft Windows |
| KB890830 | VaultStream 2016 | Windows Malicious Software Removal Tool x64 - v5.94 |
| KB5006669 | VaultStream 2016 | 2021-10 Cumulative Update for Windows Server 2016 for x64-based Systems |
| KB5006065 | VaultStream 2016 | 2021-10 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64 |
| VaultStream v10.4 Updates – Windows® Server 2019 | | |
| KB5006765 | VaultStream 2019 | 2021-10 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 |
| KB5006672 | VaultStream 2019 | 2021-10 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems |
| KB5005540 | VaultStream 2019 | Update for Microsoft Windows |
| KB890830 | VaultStream 2019 | Windows Malicious Software Removal Tool x64 - v5.94 |
| nCare MX v10.x | | |
| KB5006065 | nCare MX | 2021-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB5006669 | nCare MX | 2021-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |

| ES4K v10.x ES4K Controller (a.k.a. BlizCon) | | |
|--|--------------------------|---|
| KB5006065 | ES4K Controller | 2021-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB5006669 | ES4K Controller | 2021-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| ES4K v10.x ES4K Video Manager (a.k.a. nCare RX2) | | |
| KB5006065 | ES4K Video Manager | 2021-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB5006669 | ES4K Video Manager | 2021-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| ES4K v10.x – LiveStream StatusBoard 2 | | |
| KB5006672 | LiveStream StatusBoard 2 | 2021-10 Cumulative Update for Windows 10 Version 1809 for x64-based Systems |
| KB5006765 | LiveStream StatusBoard 2 | 2021-10 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64 |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus (Version 1.351.548.0) |
| MedPresence Tower v10.3 | | |
| KB5006065 | MedPresence Tower | 2021-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB5006669 | MedPresence Tower | 2021-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| PHP Updates | | |
| 7.1.33 and 7.4.16 – shipping versions. 7.3.30 is available upon request for v10.2.2 VaultStreams only. 7.4.23 is available upon request for v10.4 VaultStreams only. | | |

For further questions concerning this notice, please call Technical Support at 1-800-848-9024