

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD and ES4K, nCare™ MX/RX, EasyLink™ Router, LiveStream™ StatusBoard Engine, and VaultStream™ systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream™ systems.

Products: EasySuite™ HD, ES4K and nCares™, connected to a network and VaultStream™, as well as the LiveStream™ StatusBoard Engine.

Date: February 15, 2022

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

February 2022 Updates		
KB Number	Platform	Title
VaultStream™ v10.x Updates – Windows® Server 2012		
KB5010419	VaultStream 2012	Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.98
KB5010583	VaultStream 2012	Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.359.32.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.405.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.399.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.353.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.335.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.302.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.259.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.217.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.182.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.169.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.161.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.134.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.99.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.84.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.45.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.357.11.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2929.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2900.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2885.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2866.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2814.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2795.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2792.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2765.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2745.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2701.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2657.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2637.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2619.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2596.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2568.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2554.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2520.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2517.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2459.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2422.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2398.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2366.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2340.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2330.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2298.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2288.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2245.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2231.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2198.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2169.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2162.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2132.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2114.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2072.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2028.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.355.2020.0)
VaultStream™ v10.x Updates – Windows® Server 2016		
KB5010359	VaultStream 2016	2022-2 Cumulative Update for Windows Server 2016 for x64-based Systems
KB5010460	VaultStream 2016	2022-02 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.98
VaultStream™ v10.4 Updates – Windows® Server 2019		
KB5010580	VaultStream 2019	2022-2 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64
KB5010351	VaultStream 2019	2022-2 Cumulative Update for Windows Server 2019 (1809) for x64-based
KB890830	VaultStream 2019	Windows Malicious Software Removal Tool x64 - v5.98

KB4052623	VaultStream 2019	Update for Microsoft Defender Antivirus antimalware platform - (Version 4.18.2201.10)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.405.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.399.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.390.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.353.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.303.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.347.0)
nCare™ MX v10.2.3		
KB5010359	nCare MX	2022-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5010460	nCare MX	2022-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
nCare™ MX v10.4		
KB5010359	nCare MX	2022-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5010460	nCare MX	2022-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
ES4K v10.4 ES4K Controller (a.k.a. BlizCon)		
KB5010359	ES4K Controller	2022-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5010460	ES4K Controller	2022-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
ES4K v10.4 ES4K Video Manager (a.k.a. nCare RX2)		
KB5010359	ES4K Video Manager	2022-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5010460	ES4K Video Manager	2022-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
ES4K v10.4 LiveStream™ StatusBoard 2		
KB5010580	LiveStream StatusBoard 2	2022-02 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64
KB5010351	LiveStream StatusBoard 2	2022-02 Cumulative Update for Windows 10 Version 1809 for x64-based Systems
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.359.158.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.359.115.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.359.59.0)

KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.419.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.368.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 4.18.2201.10)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.321.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.281.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.230.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.176.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.119.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.65.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.357.17.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2900.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2813.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2756.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2697.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2631.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2586.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2355.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2520.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2459.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2408.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2355.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2304.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2254.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2198.0)

KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2141.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.355.2091.0)
MedPresence™ Tower v10.4		
KB5010359	MedPresence Tower	2022-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5010460	MedPresence Tower	2022-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
PHP Updates		
<p>7.1.33 and 7.4.16 – shipping versions. V10.2.2 was Tested with PHP version 7.3.33 V10.4 was Tested with PHP version 7.4.27 7.3.33 is available upon request for v10.2.2 VaultStreams only. 7.4.27 is available upon request for v10.4 VaultStreams only.</p>		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024