

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD and ES4K, nCare MX/RX, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates approved for the v10.x and up VaultStream systems.

Products: EasySuite HD, ES4K, and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: July 12, 2022

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813, or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

July 2022 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows® Server 2012		
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.371.112.0)
KB5016264	VaultStream 2012	2022-07 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB5015874	VaultStream 2012	2022-07 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB5016568	VaultStream 2012	2022-07 Security and Quality Rollup for .NET Framework 3.5, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.103
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.371.102.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.371.99.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.371.60.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.371.44.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.1159.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.1153.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.1114.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.1108.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.1068.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.1059.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.1053.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.1017.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.978.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.969.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.928.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.926.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.896.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.843.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.788.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.773.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.739.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.708.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.690.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.670.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.648.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.641.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.595.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.585.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.539.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.527.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.495.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.464.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.436.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.405.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.395.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.342.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.299.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.287.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.254.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.237.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.220.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.189.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.176.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.133.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.128.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.88.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.54.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.41.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.369.13.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1883.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1868.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1823.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1819.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1798.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1772.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1736.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1725.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1694.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1662.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.367.1647.0)
VaultStream v10.x Updates – Windows® Server 2016		
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.103
KB5016058	VaultStream 2016	2022-07 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB5015808	VaultStream 2016	2022-07 Cumulative Update for Windows Server 2016 for x64-based Systems
VaultStream v10.4 Updates – Windows® Server 2019		
KB5015811	VaultStream 2019	2022-07 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems
KB5014669	VaultStream 2019	2022-06 Cumulative Update Preview for Windows Server 2019 (1809) for x64-based Systems
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.371.112.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.1159.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.371.60.0)
KB2267602	VaultStream 2019	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.371.44.0)
KB890830	VaultStream 2019	Windows Malicious Software Removal Tool x64 - v5.103
nCare MX v10.2.3		
KB5015808	nCare MX	2022-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5016058	nCare MX	2022-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
nCare MX v10.4		
KB5015808	nCare MX	2022-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5016058	nCare MX	2022-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
ES4K v10.4 ES4K Controller (a.k.a. BlizCon)		
KB5015808	nCare MX	2022-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5016058	nCare MX	2022-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
ES4K v10.4 ES4K Video Manager (a.k.a. nCare RX2)		
KB5015808	nCare MX	2022-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5016058	nCare MX	2022-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
ES4K v10.4 LiveStream StatusBoard 2		
KB5015811	LiveStream StatusBoard 2	2022-07 Cumulative Update for Windows 10 Version 1809 for x64-based Systems
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.371.120.0)

KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.371.71.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.371.16.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.1114.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.1068.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.1114.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.1011.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.952.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.906.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.852.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.788.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.723.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.662.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.607.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.539.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.468.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.419.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.365.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.304.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.251.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.204.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.147.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.103.0)
KB4052623	LiveStream StatusBoard 2	Update for Microsoft Defender Antivirus antimalware platform - (Version 4.18.2205.7)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.369.54.0)

KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.367.1896.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.367.1841.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.367.1784.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.367.1736.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.367.1682.0)
KB2267602	LiveStream StatusBoard 2	Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.367.1628.0)
KB890830	LiveStream StatusBoard 2	Windows Malicious Software Removal Tool x64 - v5.91
MedPresence Tower v10.4		
KB5015808	MedPresence Tower	2022-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5016058	MedPresence Tower	2022-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
<p>7.4.16, 7.1.33, and 5.6.40 – shipping versions. V10.2.2 was Tested with PHP version 7.3.33 V10.4 was Tested with PHP version 7.4.30 7.3.33 is available upon request for v10.2.2 VaultStreams only. 7.4.30 is available upon request for v10.4 VaultStreams only.</p>		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024