

Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD and ES4K, nCare MX/RX, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

Products: EasySuite HD iOR systems, ES4K iOR systems, and nCare recorders connected to a network and VaultStream server software, as well as the LiveStream StatusBoard Engine.

Date: August 08, 2023

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

| August 2023 Updates | | |
|--|------------------|--|
| KB Number | Platform | Title |
| VaultStream v10.x Updates – Windows Server 2012 | | |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.393.2508.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.393.2528.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.393.2579.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.393.2585.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.393.2605.0) |
| KB890830 | VaultStream 2016 | Windows Malicious Software Removal Tool x64 - v5.116 |
| KB5029653 | VaultStream 2016 | 2023-08 Security and Quality Rollup for .NET Framework 3.5, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64 |
| KB5029312 | VaultStream 2016 | 2023-08 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.43.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.47.0) |
| KB5029368 | VaultStream 2016 | 2023-08 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.92.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.117.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.147.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.201.0) |

| | | |
|--|------------------|--|
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.227.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.277.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.288.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.318.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.352.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.361.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.436.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.469.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.522.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.539.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.594.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.661.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.674.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.730.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.751.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.806.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.838.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.879.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.918.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.939.0) |
| KB2310138 | VaultStream 2016 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.395.994.0) |
| VaultStream v10.x Updates – Windows Server 2016 | | |
| KB5028952 | VaultStream 2016 | 2023-08 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64 |
| KB5029242 | VaultStream 2016 | 2023-08 Cumulative Update for Windows Server 2016 for x64-based Systems |
| KB890830 | VaultStream 2016 | Windows Malicious Software Removal Tool x64 - v5.116 |
| VaultStream v10.4 Updates – Windows Server 2019 | | |

| | | |
|---|--------------------------|---|
| KB5029647 | VaultStream 2019 | 2023-08 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 |
| KB5029247 | VaultStream 2019 | 2023-08 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems |
| KB2267602 | VaultStream 2019 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.939.0) |
| KB890830 | VaultStream 2019 | Windows Malicious Software Removal Tool x64 - v5.116 |
| nCare MX v10.2.3 | | |
| KB5029242 | nCare MX | 2023-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| KB5028952 | nCare MX | 2023-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| nCare MX v10.4 | | |
| KB5028952 | nCare MX | 2023-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB5029242 | nCare MX | 2023-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| ES4K v10.4 ES4K Controller (a.k.a. BlizCon) | | |
| KB5028952 | ES4K Controller | 2023-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB5029242 | ES4K Controller | 2023-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| ES4K v10.4 ES4K Video Manager (a.k.a. nCare RX2) | | |
| KB5028952 | ES4K Video Manager | 2023-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB5029242 | ES4K Video Manager | 2023-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| ES4K v10.4 LiveStream StatusBoard 2 | | |
| KB5029647 | LiveStream StatusBoard 2 | 2023-08 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 version 1809 for x64 |
| KB5029247 | LiveStream StatusBoard 2 | 2023-08 Cumulative Update for Windows 10 Version 1809 for x64-based Systems |
| KB890830 | LiveStream StatusBoard 2 | Windows Malicious Software Removal Tool x64 - v5.116 |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.1056.0) |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.1008.0) |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.934.0) |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.758.0) |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.687.0) |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.613.0) |
| MedPresence Tower v10.4 | | |
| KB5028952 | MedPresence Tower | 2023-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |

| | | |
|---|-------------------|---|
| KB5029242 | MedPresence Tower | 2023-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| MAJ-2533 (UCES-4) | | |
| KB5029647 | MAJ-2533 | 2023-08 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64 |
| KB5029247 | MAJ-2533 | 2023-08 Cumulative Update for Windows 10 Version 1809 for x64-based Systems |
| KB890830 | MAJ-2533 | Windows Malicious Software Removal Tool x64 - v5.116 |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.994.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.889.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.806.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.710.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.642.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.547.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.484.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.318.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.239.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.156.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.395.85.0) |
| KB4052623 | MAJ-2533 | Update for Microsoft Defender Antivirus antimalware platform - (Version 4.18.23070.1004) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.393.2486.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.393.2414.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.393.2335.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.393.2243.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.393.2135.0) |
| KB2267602 | MAJ-2533 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.393.2046.0) |
| CV Connectivity with KE v3.1 (VaultStream with Interfaces License) | | |
| KB5028166 | KE v3.1 | 2023-07 Cumulative Update for Windows 10 Version 22H2 for x64-based Systems |
| KB5028937 | KE v3.1 | 2023-07 Cumulative Update for .NET Framework 3.5, 4.8 and 4.8.1 for Windows 10 Version 22H2 for x64 |

| | | |
|--|---------|--|
| KB890830 | KE v3.1 | Windows Malicious Software Removal Tool x64 - v5.115 |
| KB5028228 | KE v3.1 | 2023-07 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems |
| KB5028941 | KE v3.1 | 2023-07 Security and Quality Rollup for .NET Framework 3.5, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64 |
| KB5028169 | KE v3.1 | 2023-07 Cumulative Update for Windows Server 2016 for x64-based Systems |
| KB5028854 | KE v3.1 | 2023-07 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64 |
| KB5028168 | KE v3.1 | 2023-07 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems |
| KB5028936 | KE v3.1 | 2023-07 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 |
| PHP Updates | | |
| V10.2.2 was Tested with PHP version 7.3.33 V10.4 was Tested with PHP version 7.4.30 7.3.33 is available upon request for v10.2.2 VaultStreams only. 7.4.33 is available upon request for v10.4 VaultStreams only. | | |

For further questions concerning this notice, please call Technical Support at 1-800-848-9024

Microsoft and Windows are trademarks of the Microsoft group of companies.



Gyrus ACMI, Inc.
 One 9600 Louisiana Ave. North, Brooklyn Park, MN 55445 U.S.A.
www.olympusamerica.com/olympus-systems-integration