

Microsoft Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES-IP, ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD and ES-IP and ES4K iOR systems, nCare MX/RX recorders, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems and its KE Modules.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

Products: ES-IP and ES4K iOR systems and nCare recorders, connected to a network and VaultStream server software, as well as the LiveStream StatusBoard Engine.

Date: January 9, 2024

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

| December 2023 Updates | | |
|--|------------------|---|
| KB Number | Platform | Title |
| VaultStream v10.x Updates – Windows Server 2012 | | |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1944.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1928.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1887.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1858.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1833.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1823.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1783.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1780.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1774.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1944.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1737.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1732.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1687.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1675.0) |

| | | |
|-----------|------------------|---|
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1650.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1630.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1611.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1574.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1562.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1516.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1473.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1460.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1419.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1413.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1405.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1369.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1363.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1321.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1311.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1276.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1254.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1232.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1203.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1183.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1150.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1130.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1089.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1079.0) |

| | | |
|-----------|------------------|---|
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1045.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1035.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.1028.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.990.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.983.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.936.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.926.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.870.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.846.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.810.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.800.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.761.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.756.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.693.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.667.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.649.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.636.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.619.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.588.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.577.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.535.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.523.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.485.0) |

| | | |
|--|------------------|---|
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.462.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.415.0) |
| KB2310138 | VaultStream 2012 | Security Intelligence Update for Microsoft Security Essentials - (Version 1.403.404.0) |
| VaultStream v10.x Updates – Windows Server 2016 | | |
| KB2267602 | VaultStream 2016 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.403.1950.0) |
| KB5033910 | VaultStream 2016 | 2024-01 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64 |
| KB5034119 | VaultStream 2016 | 2024-01 Cumulative Update for Windows Server 2016 for x64-based Systems |
| KB890830 | VaultStream 2016 | Windows Malicious Software Removal Tool x64 - v5.120 |
| VaultStream v10.4 Updates – Windows Server 2019 | | |
| KB890830 | VaultStream 2019 | Windows Malicious Software Removal Tool x64 - v5.120 |
| KB5034273 | VaultStream 2019 | 2024-01 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows Server 2019 for x64 |
| KB5034127 | VaultStream 2019 | 2024-01 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems |
| nCare MX v10.2.3 | | |
| KB5034119 | nCare MX | 2024-01 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| KB5033910 | nCare MX | 2024-01 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB2267602 | nCare MX | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0). |
| KB2267602 | nCare MX | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0). |
| nCare MX v10.4 | | |
| KB5034119 | nCare MX | 2024-01 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| KB5033910 | nCare MX | 2024-01 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB2267602 | nCare MX | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0). |
| KB2267602 | nCare MX | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0). |
| ES4K v10.4 ES4K Controller (a.k.a. BlizCon) | | |
| KB5034119 | ES4K Controller | 2024-01 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| KB5033910 | ES4K Controller | 2024-01 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB2267602 | ES4K Controller | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0). |
| KB2267602 | ES4K Controller | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0). |

| ES4K v10.4 ES4K Video Manager (a.k.a. nCare RX2) | | |
|---|--------------------------|---|
| KB5033910 | ES4K Video Manager | 2024-01 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64 |
| KB5034119 | ES4K Video Manager | 2024-01 Cumulative Update for Windows 10 Version 1607 for x64-based Systems |
| KB2267602 | ES4K Video Manager | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0). |
| KB226760 | ES4K Video Manager | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0). |
| ES4K v10.4 LiveStream StatusBoard 2 | | |
| KB5034127 | LiveStream StatusBoard 2 | 2024-01 Cumulative Update for Windows 10 Version 1809 for x64-based Systems |
| KB5034273 | LiveStream StatusBoard 2 | 2024-01 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64 |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0) |
| KB2267602 | LiveStream StatusBoard 2 | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0) |
| MedPresence Tower v10.4 | | |
| KB5034119 | MedPresence Tower | 2024-01 Cumulative Update for Windows 10 Version 1809 for x64-based Systems |
| KB5033910 | MedPresence Tower | 2024-01 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64 |
| KB2267602 | MedPresence Tower | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.3887.0) |
| KB2267602 | MedPresence Tower | Security Intelligence Update for Microsoft Defender Antivirus - (Version 1.391.1963.0) |
| ES-IP iOR System | | |
| KB5034274 | ES-IP iOR system | 2024-01 Cumulative Update for .NET Framework 3.5,4.8 and 4.8.1 for Windows 10 Version 21H2 for x64 |
| KB5034122 | ES-IP iOR system | 2024-01 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems |
| KB890830 | ES-IP iOR system | Windows Malicious Software Removal Tool x64 -v5.120 |
| KB5033733 | ES-IP iOR system | 2024-01 .NET 6.0.26 Security Update for x64 Client |
| MAJ-2533 (UCES-4) | | |
| KB5034273 | MAJ-2533 | 2024-01 Cumulative Update for .NET Framework 3.5, 4.7.2 and 4.8 for Windows 10 Version 1809 for x64 |
| KB5034127 | MAJ-2533 | 2024-01 Cumulative Update for Windows 10 Version 1809 for x64-based Systems |
| CV Connectivity with KE v3.1 (VaultStream with Interfaces License) | | |
| KB5033372 | KE v3.1 | 2023-12 Cumulative Update for Windows 10 Version 22H2 for x64-based Systems |
| KB5033373 | KE v3.1 | 2023-12 Cumulative Update for Windows Server 2016 for x64-based Systems |
| KB5033371 | KE v3.1 | 2023-12 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems |

PHP Updates

V10.2.2 was Tested with PHP version 7.3.33

V10.4 was Tested with PHP version 7.4.30

7.3.33 is available upon request for v10.2.2 VaultStreams only.

7.4.33 is available upon request for v10.4 VaultStreams only.

For further questions concerning this notice, please call Technical Support at 1-800-848-9024

Microsoft and Windows are trademarks of the Microsoft group of companies.



Gyrus ACMI, Inc.

One 9600 Louisiana Ave. North, Brooklyn Park, MN 55445 U.S.A.
www.olympusamerica.com/olympus-systems-integration